

Copy Denture Offering

Default Design Standards



System Step	Design Step	Description
Prepare	Model Creation (IOS)	All scans to be refined.
	Tooth Removal	N/A
	Post Dam	No post dam. ONLY if request it in design codes.
	Repair Model	Any impression material will be removed from scan In the case of extra/missing material/data compromising the design, the denture it will be placed on hold.
Segmentation	Teeth Identification	
	Segmentation	
Anatomy Design	Connectors	Setting connectors to minimum: 70,70,60
	Sculpt Anatomy	Occlusion adapted at 0.0mm. Using the articulator we will adapt to 0.05mm on any contacts needing balanced occlusion. Single cases will be adapted by hand with the morphing tool when there is deep contacts.
	Sculpt Denture Base	Festooning Characterization depending on codes. Default: MEDIUM.
		Papillas to be in the middle third.
		Gingival margins to follow green line of teeth libraries (length). If no code selected: no stippling, no T-Bars.
	Coupling mechanism	Settings to be maintained
	Pre-Manufacturing	Glue Space: 0.15mm.